Running Catalyst* SmartQC* Control on your Catalyst One* Chemistry Analyzer or Catalyst Dx* Chemistry Analyzer

Please complete the following procedure on a monthly basis.

1. Clean the internal components of your Catalyst analyzer (for detailed instructions, see your analyzer operator's guide at idexx.com/product-info).
2. Tap the **Catalyst One** or **Catalyst Dx** icon on the IDEXX VetLab* Station.
3. For Catalyst One users:
 - a. Tap **Maintenance**.
 - b. Tap **SmartQC**.
 - c. Tap **Run SmartQC**.
 - d. Follow the on-screen instructions for loading the SmartQC materials and completing the run.
IMPORTANT: Catalyst SmartQC should not be run with any samples or other CLIPs/slides.

OR

For Catalyst Dx users:

- a. Tap **SmartQC**.
- b. Tap **Run SmartQC**.
- c. Follow the instructions on the Catalyst Dx touch screen for loading the SmartQC materials and completing the run.
IMPORTANT: Catalyst SmartQC should not be run with any samples or other CLIPs/slides.

You will receive pass/out of range results in less than 15 minutes.

© 2024 IDEXX Laboratories, Inc. All rights reserved. • 06-0040649-00
*Catalyst, SmartQC, Catalyst One, Catalyst Dx, and IDEXX VetLab are trademarks or registered trademarks of IDEXX Laboratories, Inc. or its affiliates in the United States and/or other countries.

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© 2024 IDEXX Laboratories, Inc. Wszelkie prawa zastrzeżone. • 06-0040649-00