

# Staining Slides for Cytology



1. **Dip** 3 times.
2. **Immerse** 30 seconds.
3. **Tap** slide edge against absorbent material.



1. **Dip** 3 times.
2. **Immerse** 5 seconds.
3. **Tap** slide edge against absorbent material.



1. **Dip** 3 times.
2. **Immerse** 5 seconds.
3. **Tap** slide edge against absorbent material.



1. **Dip** 5–10 times to remove excess stain.



1. **Dip** 5–10 times to remove excess stain.
2. **Air-dry** using the “cool” fan setting.

**Note:** Glass submission to the IDEXX Reference Laboratories, please submit at least one unstained slide. Recommend prestaining a slide for quality screening prior to submission.